



CLICKER DISPENSER OPERATION

F2000 Compomer Primer/Adhesive in the 3M™ Clicker™ Dispenser



1



2



3



4

Precautions:

- Do not depress the dispenser lever during cap removal and/or replacement.

- Do not advance the dispenser plunger during cap removal and/or replacement.

Remove Cap:

- Press and hold cap lever to unlock cap.

- Slide cap off dispenser.



5



6



7



8

Dispense:

- Touch cartridge tips to bottom of well. **Contact of the tips with the well bottom is critical to assure proper dispensing.**

- Fully depress lever to dispense material from each side of cartridge into well.
- Release lever.
- Remove dispenser from well.

Clean:

- Wipe the cartridge tips with an alcohol-dampened gauze.

Replace Cap:

- Hold the sides of the cartridge and slide cap into place until securely latched as indicated by an audible click.

3M Technical Hotline/MSDS Information 1-800-634-2249

3M Dental Products Division

St. Paul, MN 55144-1000

© 3M 1998

44-0019-9245-0

Please refer to instructions for more detailed information as well as precautionary and warranty information.