## 3M ESPE

## Adper™ Easy Bond

### Self-Etch Adhesive

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### Adhésif auto-mordancant

### I Adesivo self-etching

### Adhesivo autograbante

### Adesivo auto-condicionant

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

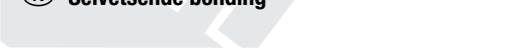
3M ESPE
Dental Products
St. Paul, MN 55144-1000

3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.

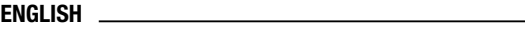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3M, ESPE, Adper, Alkaliner, Ketac, RelyX and Vitrebond are trademarks of 3M or 3M ESPE AG.
© 3M 2007. All rights reserved.

**Product Description**
Adper™ Easy Bond, manufactured by 3M ESPE, is a light-curing self-etch adhesive. It is used in combination with light-curing composite or compomer filling materials, cements and core build-up materials. The adhesive can be polymerized using halogen, LED, or plasma curing lights. Two delivery options are available: bottles for free dosing and L-Pop blisters for single use.

⚠ These instructions for use must be kept for reference for the duration of product use.

### Indications

- All classes of fillings (according to Black) with light-curing composite or compomer
- Cementation of indirect restorations made of composite or compomer, ceramic, and metal using RelyX™ ARC Adhesive Resin Cement, manufactured by 3M ESPE.
- Core build-ups made of light-curing composite
- Root surface desensitization
- Repair of composite or compomer fillings
- Intraoral repair of existing composite, porcelain fused to metal, and all ceramic restorations

Contraindications:

- Cementation of veneers and adhesive cementation of self-curing and dual-curing composites

### Precautionary Measures

#### During Use

- A rubber dam is recommended for processing the adhesive. Superficial cements of the gingiva, which may occur in some cases, are reversible.
- Temporary cements should be seated with non-eugenol temporary cements, as eugenol containing temporary cements can adversely affect the setting of Adper Easy Bond.
- Pulp Protection:** To prevent pulpal irritation, cover areas in close proximity to the pulp by applying small amounts of hard-setting calcium hydroxide material (e.g., Alkaliner™, manufactured by 3M ESPE).

#### For Patients

Adper Easy Bond contains substances that may cause allergic reactions by skin contact in certain individuals. Avoid use of this product in patients with known acrylate allergies.
If prolonged contact with oral tissue occurs, flush with large amounts of water. If an allergic reaction occurs, remove the product, and discontinue future use; seek medical attention as needed.

#### For Dental Personnel

Adper Easy Bond contains substances that may cause allergic reactions by skin contact in certain individuals. To reduce the risk of allergic response, avoid exposure to these materials; in particular, avoid exposure to uncured materials. If skin contact occurs, wash skin with soap and water.
To protect the patient's eyes and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If the liquid contacts the glove, remove and discard the gloves. Wash the hands immediately with soap and water and re-glove.
If an allergic reaction occurs, seek medical attention as needed.

Adper Easy Bond is flammable.
3M ESPE MSDSs can be obtained at www.mm.com or from your local subsidiary.

### Preparatory Steps

#### For Direct Restorations and Core Build-Ups

- Thoroughly remove loose preparation debris by spraying with water.
- Dry the cavity with gentle stream of air free of water and oil, or by blotting with cotton pellets. Do not overdry!

#### Before Cementing Indirect Restorations with RelyX ARC

► Completely remove any temporary cement remaining on the cavity and the restoration.

► Prepare the restoration for cementation as is needed for the respective material.

- Dry the cavity with gentle stream of air free of water and oil, or by blotting with cotton pellets. Do not overdry!

#### For Uncut Enamel

- Clean the enamel as usual (e.g., with polishing paste or powder blasting), and thoroughly rinse with spraying with water.
- Ech for 15 sec with a commonly used phosphoric acid etching gel; then thoroughly rinse again.

#### Before Desensitization of Root Surfaces

- Clean the root surface as usual (e.g., with polishing paste or powder blasting), thoroughly rinse by spraying with water, and dry with air free of water and oil.

#### For the Repair of Fillings or Restorations

- Roughen the composite or compomer filling, or the composite or ceramic veneer respectively, with ceramic reamer the glaze 1 mm beyond the margin.
- Thoroughly clean the surface with water and dry it with air.
- Silante ceramic veneers using RelyX™ Ceramic Primer, manufactured by 3M ESPE. Cover any visible metal using Masking Agent, manufactured by 3M ESPE, after applying Adper Easy Bond Self-Etch Adhesive. Please refer to the Instructions for Use of the respective product during processing.

#### Cavity Liner/Base

Self-adhesive materials such as glass ionomer cements (e.g., Ketac™ Bond, manufactured by 3M ESPE) or resin-modified glass ionomers (e.g., Vitrebond™, manufactured by 3M ESPE) can be used as a cavity liner or as a base.

After applying Adper Easy Bond, flowable composites and flowable compomers can be used as cavity liner.

#### Dosing from the Vial

- Unscrew the cap of the vial.
- Dose the necessary amount of Adper Easy Bond to a mixing well and protect the liquid from light.
- Carefully close the vial tightly after dosing. If necessary, disinfect the vial in accordance with local hygiene regulations.

#### Dosing from the L-Pop Blister

- Hold the shaft of the disposable applicator without disposable applicator.
- Hold the shaft of the disposable applicator in one hand, covering the blister opening with your thumb. This will prevent the etching liquid from splashing out.
- With the thumb and index finger of the other hand, squeeze the reservoir of the L-Pop blister, starting with the outer end, in the direction of the disposable applicator.

- Turn the disposable applicator back and forth in the liquid to wet it completely.
- For easy application in the cavity, the disposable applicator can be bent while it is still in the L-Pop blister. In order to do so, only pull the disposable applicator out of the blister far enough so that the narrow section of the applicator shaftt can be seen. Fold the applicator shaftt at this location over your thumbnail.
- During treatment, move the liquid into the open end by fully squeezing out with the thumb and index finger.

#### Application

- Avoid any contamination with blood, saliva, or sulcus fluid during application. A rubber dam is recommended to keep the treated area sufficiently dry.
- Apply the adhesive with the disposable applicator for 20 sec to all surfaces of the cavity.
- Rewet the disposable applicator as needed during application. Avoid contact of the adhesive with mucosal tissue.
- Subsequently, air the liquid for approx. 5 sec until the film no longer moves, indicating complete vaporisation of the solvent.
- Cure the adhesive with a commonly used curing light for 10 sec.
- Depending on the indication, continue to work with the desired composite or compomer material as indicated in the pertinent Instructions for Use.

#### Disposal

Only dispose of completely empty packages. The residue of the etching fluids – if heavily diluted with water or neutralized – can be poured down the drain.

#### Storage and Shelf Life

To achieve the maximum amount of shelf life, store the product at 2–8°C/ 36–46°F. Refrigeration is not required if the product is depleted within six months. Do not use after the expiration date.

#### Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

#### Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for your's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

#### Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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#### Applikation

- Avoid any contamination with blood, saliva, or sulcus fluid during application.
- Apply the adhesive with the disposable applicator for 20 sec to all surfaces of the cavity.
- Rewet the disposable applicator as needed during application. Avoid contact of the adhesive with mucosal tissue.
- Subsequently, air the liquid for approx. 5 sec until the film no longer moves, indicating complete vaporisation of the solvent.
- Cure the adhesive with a commonly used curing light for 10 sec.
- Depending on the indication, continue to work with the desired composite or compomer material as indicated in the pertinent Instructions for Use.

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