## ENGLISH

Express™ registration material, manufactured by 3M ESPE, is a hard setting vinyl polysiloxane material designed for making precise bite registration impressions, as an impression mold for fabricating temporary restorations, or as a fast-setting putty for the two step putty/wash technique.

The following table describes seating time and oral setting time.

Seating Time* ≤ 30 Sec	Oral Setting Time 1.5 Min.
---------------------------	-------------------------------

\*From start of putty mix.

## DIRECTIONS FOR USE

### Bite Registration Impressions

1. Dispense equal volumes of registration base and catalyst.
2. Vigorously mix\* base and catalyst with fingertips until a uniform color is achieved (20 seconds).

**\*CAUTION: Latex gloves will inhibit the setting of the material when mixed when gloves are worn! Test gloves to confirm proper setting times, wear vinyl gloves or use vinyl overliners. Alternatively, remove gloves and thoroughly wash and dry hands to remove sources of potential contamination (e.g., glove residues and hand lotions).**

3. Shape the mixed material into a cylinder or rope and place half onto the occlusal surface of each side of the mouth.

**Note:** The material can also be used in conjunction with a bite registration tray. Guide the patient into desired occlusion and immobilize for 1.5 minutes until set.

4. Carefully remove the registration from the patients mouth. Rinse the registration under water and dry.

5. Trim the registration with a scalpel to remove any material that would interfere with the soft tissue or interproximal areas that would inhibit proper seating of the models.

6. Disinfect the registrations by placing them in any ADA accepted liquid disinfectant for the appropriate length of time. Remove and rinse the bite registration in water and blow dry.

7. If the bite registrations are to be sent to a dental laboratory, they should be shipped in packaging designed to prevent distortion and contamination.

## STORAGE AND USE

- Registrations should be stored at dry room temperatures. Do not store in water or excessive humidity.
- Shelf life at room temperature is 24 months. See outer package for expiry date.
- The system is designed to be used at room temperature of approximately 21-24°C or 70-75°F.
- Avoid contaminating the surfaces to be recorded with chemicals that could contaminate and inhibit the setting (e.g. acrylic and methacrylate residues, latex rubber, and sulfur compounds).

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

## WARRANTY

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

## LIMITATION OF LIABILITY

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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3M, ESPE and Express are trademarks of 3M or 3M ESPE AG.
© 3M 2002. All rights reserved. 44-0023-3912-3-A