

Introducing the compomer that's easy to handle.  
**3M™ F2000 Compomer Restorative System**



**3M** *Reliability*

# 3M Dental expertise creates a new compomer.

## *Superior handling for fast, easy placement.*

3M Dental research and development has taken the compomer concept – combining the positive attributes of a glass ionomer with those of a composite – to a new level. After years of experience with both glass ionomer and composite technology, producing this compomer was a natural extension of 3M Dental's expertise.

The easy-to-use 3M™ F2000 Compomer Restorative is a one-part, light-curable filling material designed to be used with 3M™ Single Bond Dental Adhesive. 3M F2000 compomer restorative offers dentists a fluoride-releasing restorative with superior handling characteristics that provide for fast and easy placement.

### **Meets your needs for a variety of indications.**

The 3M F2000 compomer restorative system is indicated for restoring a number of cavity classifications in adult and primary teeth. Indications include:

- Class III and V cavities
- Class I and II cavities in primary teeth
- Cervical erosion/abrasion lesions
- Temporary repair of fractured teeth
- Laminate or open sandwich technique
- Core buildup where at least half of the coronal tooth structure is remaining to provide support for the crown


### **Superior handling compared to the leading compomer.**

3M F2000 compomer restorative is an easy-to-use system that offers improved handling characteristics that save you time:

- Does not stick to instruments
- Does not slump
- Adapts well to margins
- Easy to shape and contour
- Easy to polish

## *Direct Compomer Restorative Technique*


F2000 compomer restorative with 3M Single Bond adhesive system used in a Class V anterior restoration. Please refer to instructions for precautionary, detailed use, storage and warranty information.

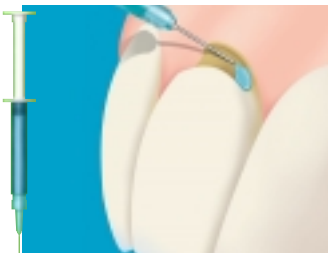

- Apply 3M™ Scotchbond™ etchant to enamel and dentin for 15 seconds.
- Rinse and blot dry.




- Apply two consecutive coats of 3M Single Bond adhesive to enamel and dentin.
- Gently dry 2-5 seconds.

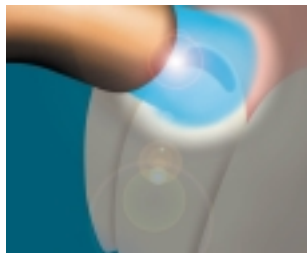

- Light cure the adhesive for 10 seconds.



- Place F2000 compomer restorative in increments.

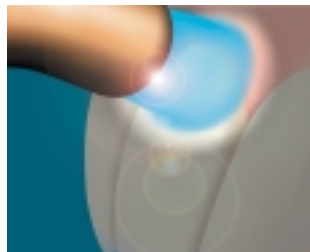

- Light cure each increment for 40 seconds.




- Use the 3M™ Sof-Lex™ Finishing and Polishing System and 3M™ Sof-Lex™ Strips for finishing and polishing the restoration.

**High level of fluoride release.**

3M F2000 compomer restorative provides a level of fluoride release not available in traditional composites. Its fluoride release is also unmatched by the leading compomer.

**Cumulative Fluoride Release**

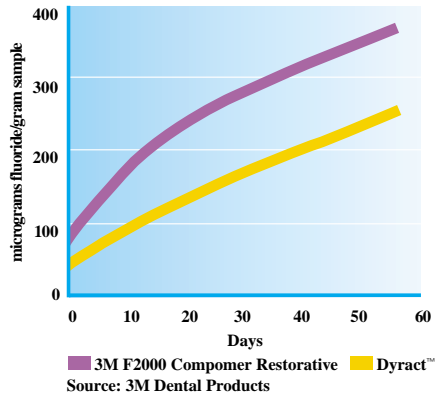

**More versatile than the competition.**

Not only does 3M F2000 compomer restorative offer you the option of unit dose (capsule), or multi-dose (syringe) delivery, it comes in 13 shades... a greater shade selection than any other compomer.

Shades include Vita™ shades A2, A3, A3.5, A4, B2, B3, C2, C4 and D3, plus four specialty shades: Pedo, CG, CY and Blue for core buildup. With so many shades and the F2000 compomer restorative shade guide, you minimize shade-matching guesswork and meet the needs of most patients.

Vita™ Shade	3M F2000
A2	✓
A3	✓
A3.5	✓
A4	✓
B2	✓
B3	✓
C2	✓
C4	✓
D3	✓
<b>Additional Shades</b>	
Pedo	✓
CG	✓
CY	✓
Blue	✓

**Use 3M F2000 compomer restorative with the 3M Single Bond adhesive system.**

- Provides excellent bond strengths to dentin and enamel.
- Bottle can be flipped open with one hand, dispenses the precise number of uniform drops needed, and snaps securely shut. It even lets you see how much is left inside.
- Technique is simple and has fewer steps.

**Feature for feature, you can't find a better compomer than 3M F2000 Compomer Restorative.**

- Superior handling compared to the leading compomer
- Greater wear resistance than the leading compomer and hybrid or conventional glass ionomers
- A higher level of fluoride release than the leading compomer
- Offers excellent esthetics; blends well with the tooth
- Excellent bond strengths when used with the 3M Single Bond dental adhesive



# Ordering Information



## 3M™ F2000 Compomer Restorative System Introductory Syringe Kit

- | Order No. | Description  |
|-----------|--|
| 2022      | <p>Kit containing:</p> <ul style="list-style-type: none"> <li>• 5 - 4gm syringes of 3M F2000 Compomer Restorative (one each of five Vita™ shades: A2, A3, A4, C2, D3)</li> <li>• 6ml bottle of 3M Single Bond Dental Adhesive</li> <li>• 3ml etchant syringe and tips</li> <li>• shade guide, one-hole mix well and mixing pad</li> <li>• brush handle with 100 fiber tips</li> <li>• technique guides and instructions</li> </ul> |

### Refills

- | Order No. | Description  |
|-----------|--|
| 2020      | <p>Syringe Refill</p> <ul style="list-style-type: none"> <li>• 1 - 4gm syringe</li> <li>• Available in 13 shades: A2, A3, A3.5, A4, B2, B3, C2, C4, D3, Pedo, CG, CY and Blue</li> </ul> |
| 3411      | <p>3M Single Bond Dental Adhesive Refill</p> <ul style="list-style-type: none"> <li>• 6ml adhesive, instructions</li> </ul>  |



## 3M™ F2000 Compomer Restorative System Introductory Capsule Kit

- | Order No. | Description  |
|-----------|--|
| 2023      | <p>Kit containing:</p> <ul style="list-style-type: none"> <li>• 60 - 0.20gm capsules of 3M F2000 Compomer Restorative (10 each of six Vita™ shades: A2, A3, A4, B2, C2, D2)</li> <li>• 6ml bottle of 3M Single Bond Dental Adhesive</li> <li>• 3ml etchant syringe and tips</li> <li>• shade guide and one-hole mix well</li> <li>• brush handle with 100 fiber tips</li> <li>• technique guides and instructions</li> </ul> |

- | Order No. | Description   |
|-----------|---|
| 2021      | <p>Capsule Refill</p> <ul style="list-style-type: none"> <li>• 20 - 0.20gm capsules</li> <li>• Available in 13 shades: A2, A3, A3.5, A4, B2, B3, C2, C4, D3, Pedo, CG, CY and Blue</li> </ul> |
| 3411      | <p>3M Single Bond Dental Adhesive Refill</p> <ul style="list-style-type: none"> <li>• 6ml adhesive, instructions</li> </ul>   |

**3M Technical Hotline: 1-800-634-2249**

**3M Dental Products Web Site: <http://www.mmm.com/dental>**



### Dental Products

3M Center, Building 275-2SE-03  
St. Paul, MN 55144-1000  
1-800-634-2249

### Dental Products 3M Canada, Inc.

Post Office Box 5757  
London, Ontario, Canada N6A 4T1  
1-800-265-1840, ext. 6229



Recycled Paper  
40% pre-consumer,  
10% post-consumer

3M and Sof-Lex are trademarks of 3M.

Dyract is a trademark of Caulk®/Dentsply®.

VITA™ is a trademark of VITA ZAHNFABRIK Bad Sackingen, Germany.

Printed in U.S.A.  
© 3M, 1997  
70-2008-7940-4