Impregum™ Penta™ Soft Heavy Body

Impregum™ Penta™ Soft Light Body

Polyether Impression Material

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3M ESPE
Dental Products
St. Paul, MN 55144-1000

3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.

3M, ESPE, Garant, Impregum, Impresept, Penta, PentaMatic and Pentamix are trademarks of 3M or 3M ESPE AG.
© 3M 2006. All rights reserved.

44000737519/03

ENGLISH

Product Description

Impregum™ Penta™ Soft Heavy and Light Body, manufactured by 3M ESPE, are heavy-bodied and light-bodied consistency polyether impression materials, respectively, to be used with the mixing device, Pentamix™, manufactured by 3M ESPE. The mixing ratio is 5 volumes base paste : 1 volume catalyst.

Due to its lower Shore hardness, the impression material is suitable for use with impressions with slight undercuts even without additional blocking out.

The poly bags are sealed with a **PentaMatic™** sealing cap, manufactured by 3M ESPE. The **PentaMatic** sealing cap automatically opens the bag once sufficient pressure is established by the plunger of the Pentamix.

For details on Pentamix and accessories, Polyether Adhesive, Penta™ Elastomer syringe, Penta™ Mixing Tip Red and Impregum™ Garant™ Soft Light Body, all products manufactured by 3M ESPE, please refer to the corresponding Instructions for Use.

⚠ These Instructions for Use should not be discarded for the duration of product use.

Areas of Application

- Impression of inlay, onlay, crown, bridge and veneer preparations
- Functional impressions
- Fixation and implant impressions

Preparation

► Block-out undercuts or areas where gingival recession exists to prevent the impression material from "locking" onto the tooth structure. Failure to block-out may make tray removal difficult, or cause extraction of natural teeth or prosthesis.

Impression trays:

All impression trays generally used for precision impressions are suitable.
► For sufficient adhesion, apply a thin layer of Polyether Adhesive to the tray and allow to dry completely (at least 30 – 60 sec, 15 min drying time are optimal).

Pentamix/Penta Soft cartridge/poly bag:

- **Place the Impregum Penta Soft Heavy and Light Body materials in the corresponding designated cartridges only!**
- Poly bags equipped with a **PentaMatic** sealing cap must be used only with Penta Mixing Tips Red, manufactured by 3M ESPE.
- Discard the first 3 cm of paste extruded from newly filled cartridges prior to the first use for impression taking. The color of the dispensed paste must be uniform.
- If a new mixing tip is installed when the cartridge is inserted, check whether the drive shaft is engaged before you start mixing.

Retraction

Solutions based on aluminium hydroxide chloride or aluminum sulphate are suitable retraction agents.

Retraction with threads or rings containing epinephrine (adrenaline), 8-hydroxyquinoline sulphate or iron (III) sulphate may impair the setting of polyether impression materials.

- Areas from which impressions are to be taken should be kept dry.
- In subgingival preparations, hemostatic threads or rings may be used.
- **Prior to taking the impression, completely remove any residual retraction agent by rinsing and drying.**

Dosing and Mixing

► Dosing and mixing are performed automatically in the Pentamix device.

Times	Processing time from start of mixing* min:sec	Setting time from start of mixing* min:sec	Residence time in the mouth** min:sec
Impregum Penta Soft Heavy Body	2:30	6:00	3:30
Impregum Penta Soft Light Body	3:15	6:30	3:15

The working times shown are in conformity with ISO 4823:2000. Higher temperatures will shorten the working time, while lower temperatures will prolong it. We recommend a working time of 1:45 min:sec and compliance with the indicated setting time from the start of mixing to assure an ideal impression result at all times under clinical conditions.

* Start of mixing = entry of paste into the mixing tip.
** For product combinations, longer times are valid.

Impression Taking

One-step technique

- For application around the preparation, attach the Penta Elastomer syringe to the Penta Mixing Tip Red of the Pentamix device, and fill with Impregum Penta Soft Light Body.
- Subsequently, change cartridges and load Impregum Penta Soft Heavy Body onto the tray previously prepared with adhesive.
- While loading the tray, apply the mixture into the sulcus or cavity from bottom up, always keeping the tip of the application nozzle immersed in the material and in contact with the surface of the tooth.
 - **Depending on the tray size, initiate application around the preparation once the tray is half-loaded to ensure that the tray can be positioned in the mouth as soon as the application around the preparation is completed.** Failing this, the syringe material will set more rapidly than the tray material, leading to distortions of the impression.
- Immediately after application around the preparation, position the loaded tray in the mouth, and hold it there without applying pressure until setting is complete.
- To remedy initial adhesion ("setting the valve"), especially with upper jaw impressions, remove the tray from the gingiva on one side in a posterior position. If this proves difficult it may be necessary to carefully blow some air or water between the impression and the gingiva.

Functional Impression

- After preparing the custom tray with adhesive, load Impregum Penta Soft Heavy Body onto the tray, place the tray in the desired location in the mouth, and have the patient perform functional excursions.
- If required, use Impregum Penta Soft Light Body or Impregum Garant Soft Light Body for precise impression taking (correction).

After Impression Taking

► Thoroughly examine and explore the sulcus of the prepared teeth and surrounding dentition. Remove any residual cured impression material from the mouth.

Hygiene

- Place the impression in a standard disinfectant solution commonly used for impressions, e.g. Impresept™*, for the period of time recommended by the manufacturer, i.e. 10 min in the case of Impresept. Excessive disinfection may damage the impression.
- After disinfection, rinse the impression under running water for approx. 15 sec. * Impresept is not available in all countries.

Model Preparation

- Prepare a cast from the impression with a commercial specialized stone plaster no earlier than 30 min and no later than 14 days after taking the impression.
- To avoid introducing bubbles into the model, briefly pre-rinse with water, and dry with air before pouring.
- **Do not use surfactants** as these impair the quality of polyether impressions, and are not required!
- Polyether impressions can be silver-coated, whereas copper-coating is not feasible.

Cleaning

- Paste that has not set may be removed with ethanol or by rinsing with water and soap.
- The adhesive can be removed from metal trays with acetone.

Notes

- At temperatures below 18°C/64°F, the viscosity of the pastes may increase sufficiently to make mixing in the device difficult. After keeping the pastes at 18°C/64°F for at least one day the processability is re-established without compromising quality.
- Direct exposure to sun and moist storage conditions damage the impression.
- Polyether impressions should not be exposed to solvent-containing liquids, as this may result in swelling and imprecise modelling.
- 3M ESPE Polyether materials may only be combined with 3M ESPE polyether materials. Doing so the working and setting times of the individual products must necessarily be kept.

Precautionary Measures

If an adverse reaction is observed in a patient discontinue use of the product in this patient.

3M ESPE MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Storage and Stability

Store the product at 18–25°C/64–77°F. **Do not refrigerate!**
Do not use after the expiration date.
Store impressions dry and below 30°C/86°F in the dark.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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