## Technique for pouring and mounting Dual Arch Trays (Dual Arch tray also known as Triple Trays, Double Bite Trays, or Combined Arch Trays)

**Required Equipment:** Vacuum mixing unit and bowl, mixing spatula, die pins (medium), separating medium, carding wax (rope wax), crown and bridge articulator, die stone, lab stone, plaster (#2 fast set), die saw, small flat brush, permanent marking pen.

**Important:** Place a mark indicating the center line of the preparations with a permanent marking pen on the highest point of the buccal and lingual mucosal fold of the impression to be used as a reference point for the placement of the die pins.

1. Mix preferred die-stone according to manufacturer's instructions. Carefully pour the prepared area and adjacent teeth with die-stone. Pour enough die-stone material so that it is 10-15 mm higher than the prepared margins of the preparation.

2. Position a die pin into the wet die-stone directly above and center of the preparation. Place a second and third die pin above (and center) of the abutment teeth (mesial and distal of the prepared tooth) and allow to harden for 30 minutes. All pins should be vertical and parallel to each other. Only the serrated portion of the die pin should be submerged into the die-stone.

3. After a 30 minute set, paint the die-stone areas around the die pins with a separating medium. Place a small ball of red carding wax at the ends of each die pin (these serve as location and access points for the next procedure).

4. Mix lab stone and pour (or stack) a layer around the die pins and the remaining exposed surfaces of the set die-stone. The lab stone should reach to the top of the red wax balls (**do not cover the wax balls completely**). Allow lab stone to harden for 30 minutes. (The second layer serves as a base and index for the pinned dies).

5. Mix another quantity of lab stone and pour the opposing arch. Allow lab stone to harden (usually 30-45 minutes). At this point, the upper and lower

9. Using a die saw, cut vertically between the preparation and the abutment tooth (mesially and distally). The cut should terminate at the junction of the die stone and lab stone layer (that had been prepared with a separating medium). The die can be easily removed for further trimming.

10. Carefully trim the die stone away to expose the margins. Mark the margins with a red pen. Lubricate the die and prepare for fabrication of the restoration.

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arches should be poured with the impression tray intact. **DO NOT SEPARATE.**

6. Attach the upper and lower arches to a suitable articulator with a mixture of fast set plaster. Use enough plaster so that it can be smoothed (for handling and stability of the poured models when the impression is removed). Allow the plaster to harden (10-15 minutes).

7. Grasp the maxillary and mandibular member of the articulator simultaneously, and **carefully** pull apart the two arches from the impression material and tray. Check the bite at this time.

8. Trim away all sharp portions of the set stone and plaster.

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