Imprint™ II Quick Step

Vinyl Polysiloxane HeavyBody/Light Body Impression Material

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Vinyl Polysiloxan Heavy Body/Light Body Avtrycksmaterial

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ENGLISH

Product Description

Imprint™ II Garant™ Quick Step Heavy Body/Light Body Impression Material, manufactured by 3M ESPE, is a fast, rigid setting, hydrophilic dental impression material designed for making a precise impression of an inlay, onlay, veneer, or crown. Imprint II Garant Quick Step heavy body and light body impression materials are formulated for use in a one-step/two viscosity impression technique. Imprint II Garant Quick Step heavy body is to be used in combination with Imprint II Garant™ Quick Step Regular Body or Light Body impression material, manufactured by 3M ESPE. The Imprint II Garant Quick Step heavy body impression material has 1:15 minutes of room temperature working time with an oral setting time of 2:30 minutes. The Imprint II Garant Quick Step regular and light body impression materials have 40 seconds of oral working time, and an oral setting time of 2:30 minutes.

For other impression material needs, Imprint™ II Garant™ Heavy Body/Light Body Impression Material with 1 minute of oral working time, and a 4 minute setting time may be used for most indications. If a monophase material is desired, Imprint™ II Garant™ Monophase Impression Material, manufactured by 3M ESPE, is available with 1 minute of oral working time, and a 4 minute setting time.

Indication for Use

Imprint™ II Garant Quick Step impression material has 40 seconds of oral syringing time, and it is designed for use in fast procedures where the impression tray can be seated within 40 seconds from the start of oral syringing. This material is designed for a single unit, but can be used in some cases for up to three adjacent units. **Do not syringe more than three adjacent units.** To avoid incomplete or inaccurate impressions, be careful not to exceed the 40 seconds of oral working time.

Due to the rigidity and short working time of the impression materials, the following trays are recommended.

Dual Arch - Metal, and Disposable impression trays

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IMPORTANT: PLEASE NOTE:

1. Imprint™ II Garant™ Quick Step impression material is ideally suited for a single unit preparation. To avoid incomplete or inaccurate impressions, be careful not to exceed the 40 seconds of oral working time.
2. **Do not syringe more than three adjacent preparations.**
3. Bleed both the heavy body impression material and light body impression materials before each use.
4. As with any rigid-setting impression material, it may be necessary to block-out undercuts or areas where gingival recession exists to prevent the material from "locking" on to tooth structure. Failure to block-out may make tray removal difficult, or cause extraction of natural teeth, or prosthesis.
5. Avoid surface contact with chemicals known to inhibit the set of vinyl polysiloxane impression materials (e.g. acrylic and methacrylate residues, latex rubber, and sulfur compounds). If acrylic and methacrylate residues are present, clean the affected tooth surface with isopropyl alcohol before making the impression. **To avoid inhibition caused by custom temporary materials, the final impression should be made before fabricating the provisional crown or bridge.**
6. If contamination from latex rubber or sulfur is suspected, rinse the area with an aqueous solution of hydrogen peroxide before making the impression.
7. Once the tray is seated in the mouth, Imprint II Garant Quick Step impression material requires 2:30 minutes of oral setting time. It is not necessary to add any unused working time to the oral setting time.
8. The stone model may be poured 45-120 minutes after disinfecting the impression. Imprint II Garant Quick Step impression material is very stable. A stone model may be poured up to two weeks after the impression has been made.
9. Imprint II Garant Quick Step heavy body impression material is not suitable for use as a monophase/single phase, or light body impression material.

Clinical Timing Recommendations

Imprint II Garant Quick Step Heavy Body Impression Material (Green)

Tray Filling 1:15 minutes	Oral Set Time: 2:30 Minutes
------------------------------	--------------------------------

Imprint II Garant Quick Step Regular Body Impression Material (Purple)

Intraoral Syringe Time 40 seconds	Oral Set Time: 2:30 Minutes
--------------------------------------	--------------------------------

Imprint II Garant Quick Step Light Body Impression Material (Orange)

Intraoral Syringe Time 40 seconds	Oral Set Time: 2:30 Minutes
--------------------------------------	--------------------------------

	Total Working Time	Approximate Removal from Mouth Time
Heavy Body Impression Material	@ 23°C (73°F) ≤ 1:15 minutes	2:30 Minutes
Regular & Light Body Impression Materials	@ 33.3°C (92°F) ≤ 40 seconds	2:30 Minutes

ONE-STEP/TWO VISCOSITY IMPRESSION TECHNIQUE

Recommended Materials: Imprint™ II Garant™ Quick Step Heavy Body Impression Material in combination with Imprint II Garant Quick Step Regular Body or Light Body Impression Materials

Bleed both the Imprint II Garant Quick Step heavy body impression material and Imprint II Garant Quick Step light body impression material before each use. This action will reduce the occurrence of plugging.

1. Select or prepare a rigid tray of sufficient size and wall height to accommodate a 2-3 mm thickness of impression material surrounding the tooth.
2. Place the impression tray into the patient's mouth, and verify that the patient bites into centric occlusion, does not bite onto the tray, and that no other obstructions are observed.
3. Brush a thin coat of 3M ESPE VPS tray adhesive on all surfaces of the tray that will come into contact with the impression material. Allow the tray adhesive to air-dry a minimum of 5 minutes.
4. Prepare and isolate the tooth. Place retraction cord(s).
5. Syringe both sides of the adhesive-coated tray with Imprint II Garant Quick Step heavy body impression material while keeping the tip immersed in the dispensed impression material to avoid entrapping air.

Over-fill the impression tray so that the Imprint II Garant Quick Step heavy body tray impression material flows over the prepared tooth, abutments, and into the buccal mucosal or anterior mucosal folds.

The maximum tray filling time is 1:15 minutes for Imprint II Garant Quick Step heavy body impression material.

6. Remove the retraction cord(s).

7. Syringe Imprint II Garant Quick Step regular body or light body impression material around the prepared tooth with an intraoral syringe. Syringe the light body impression material with a stirring motion, and keep the tip immersed in the wash material to avoid entrapping air, and to ensure complete coverage of the prepared surfaces.

OR

Syringe Imprint II Garant Quick Step regular body or light body impression material from the cartridge by attaching a Garant Intraoral Tip to the Garant yellow mixing tip. Use moderate pressure and push the blunt end of the Garant intraoral tip into the mix tip. A click will be heard when the intraoral tip is locked into position.

8. Carefully seat the tray to avoid contact of the prepared tooth with the tray.

Note: The tray must be seated within 40 seconds from the start of syringing the light body impression material

9. Instruct the patient to bite down. Verify that the patient has bitten down into centric occlusion. **Oral setting time is 2:30 minutes after the tray is inserted into the mouth.**

10. Apply downward pressure along the periphery of the tray to break the seal of the impression, and remove the impression from the mouth.

11. Visually inspect the impression for evidence of defects or tears. Examine and explore the sulcus of the prepared tooth and surrounding dentition, and remove any residual impression material.

12. Immediately rinse the impression with water. Place the impression into any liquid disinfectant for recommended disinfecting time. Rinse the impression in water and blow dry.

13. The stone model may be poured 120 minutes after disinfecting the impression. Impressions made using Imprint™ II Garant Quick Step impression material are very stable. A stone model may be poured up to two weeks after the impression has been taken.

14. Impressions should be shipped in packaging designed to prevent distortion and contamination.

STORAGE AND USE

• The system is designed to be used at room temperature of approximately 21-24°C (70-75° F)

• Shelf life at room temperature (see above) is 36 months. See outer package for expiration date.

• Impressions made with Imprint™ II Garant™ Quick Step impression material should be stored dry at room temperature (see above). Do not store in water or excessive humidity.

• Avoid storage conditions which are below 5° C (41° F) and 20% relative humidity, or higher than 27° C (81° F) and 80% relative humidity. Storage under these conditions may adversely affect the quality and performance of Imprint II Garant Quick Step impression material.

• Impressions made with Imprint II Garant Quick Step impression material may be silver plated or copper plated.

• Using materials that have been cooled by refrigeration may extend Imprint II Garant Quick Step impression material working time. Oral set times may be greater than the recommended 2:30 minutes, if materials are refrigerated before use.

REGULATORY SPECIFICATION DATA:

• International Standard ISO 4823

• Chemical Nature: Vinyl Polysiloxane

• Recovery from Deformation: > 99.0%

• Linear Dimensional Change: 24 Hrs. < 0.9%

• Strain in Compression:
Heavy Body Impression Material 1.65 - 2.27%
Regular Body Impression Material 3.35 - 4.03%
Light Body Impression Material 4.15- 4.67%

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Minimum: 120 minutes

Maximum: Up to 2 weeks

• Metalizing Bath: Ag, Cu

International Standard ISO 4823:

	Heavy Body Impression Material	Regular Body Impression Material	Light Body Impression Material
Mix Consistency	Type 1 Heavy Body	Type 3 Low-Light Body	Type 3 Low-Light Body
Mix Time @ 73°F	Auto-Mix	Auto-Mix	Auto-Mix
Total Working Time @ 73°F	1:15 min.	1:15 min.	1:15 min.
Min Oral Setting Time @ 92°F	2:30 min.	2:30 min.	2:30 min.

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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3M, ESPE, Imprint and Garant are trademarks of 3M or 3M ESPE AG. © 3M 2002. All rights reserved. 44-0023-3932-1-A
3M ESPE AG