3M ESPE

Ketac™ Fil Plus Aplicap™

Glass Ionomer Filling Material
Glasionomer Füllungsmaterial
Matériau d'obturation vitro-ionomère
Materiale di riempimento a base di vetroionomero
Material de relleno de ionómero vítreo
Material de obturação de ionómero vítreo
Glasionomeer vulmateriaal
Υαλοίονομερες εμφρακτικό υλικό
Glasjonomer-fyllingsmaterial
Lasi-ionomeeri-täytämateriaali
Glasionomer-fylldningsmateriale
Glassionomer fyllingsmateriale

Instructions for Use	Gebruiksinformatie
Gebrauchsinformation	Πληροφορίες χρήσεως
Mode d'emploi	Bruksanvisning
Informazioni per l'uso	Käyttöinfo
Información de uso	Brugsanvisning
Instruções de uso	Brugsanvisjon

Made in Germany by 3M ESPE AG	
Dental Products	CE
D-82229 Seefeld - Germany	0123

3M ESPE	
Dental Products	
St. Paul, MN 55144-1000	

3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.

3M, ESPE, Alkaliner, Aplicap, Capmix, Ketac and Rotomix are trademarks of 3M or 3M ESPE AG.
© 3M 2003. All rights reserved.

ENGLISH

Product Description
Ketac™ Fil Plus Aplicap™, manufactured by 3M ESPE, is a glass ionomer filling material in capsules. The following colors are available: A1/bright, A2/bright-yellow, A3/yellow, A3,5/brown, A4/yellow-brown, C2/gray, C3/gray-brown, DGO/dark-gray. Minimally 0.11 ml of the material are dispensable from each capsule. For details on all products mentioned below please refer to the corresponding Instructions for Use. Instructions for Use should not be discarded for the duration of product use.

Areas of Application
Class III and V fillings
Wedge-shaped defects
Class I small fillings
Primary tooth fillings
Restorative fillings prior to crown preparation
Fissure sealing

Preparation
Remove the carious substance only, no undercuts required. Do not prepare thin edges since the material requires a wall thickness of 0.5 mm.

Pulp protection
In order to prevent irritations of the dental pulp, areas close to the pulp should be covered punctiformly with a calcium hydroxide preparation (e.g. Alkaliner™, manufactured by 3M ESPE).

Conditioning
For optimal chemical bonding to the teeth the smear layer produced in the preparatory steps must be carefully removed by applying Ketac Conditioner, manufactured by 3M ESPE, to the prepared surfaces and allowing the substance to react for 10 sec.

Then rinse with copious amounts of water.
Blow the cavity dry in only 2–3 short intervals with water and oil-free air or dab dry with cotton pellets. Do not over-dry! **The cavity should be just so dry that the surface has a matt shiny appearance.** Excessive drying can result in post-operative sensitivity after filling.
Avoid further contamination.

Activation Mixing Application
Activate the capsules with Aplicap™ activator for 2 sec, then mix using a high frequency mixing device (at approx. 4300/min), e.g. Capmix™, or the rotary mixer, Rotomix™, and apply with the Aplicap™ Applier, all products manufactured by or for 3M ESPE. Carefully avoid contamination during application.
Aplicap Activator and Applier are labeled red.

Attention
Avoid any lag times between the processes of activation, mixing, and application, as the material is in the process of setting which may impair or prevent application of the material. Ketac Fil Plus Aplicap sticks to metal instruments and should be washed off with cold water prior to setting.

Times

At an ambient temperature of 23°C/73°F the following times apply:	
	min : sec
Activation of capsule	0 : 02
Mixing with Rotomix	0 : 08
or	
Mixing with high frequency mixing device such as Capmix	0 : 10
Processing from start of mixing	1 : 30
Setting from start of mixing	7 : 00

At ambient temperatures above 23°C/73°F the setting process is accelerated. Exceeding the indicated processing time may lead to reduced bonding to enamel and dentin.

Shorter mixing times prolong the processing and setting times, whereas longer mixing times shorten the processing and setting times. The mixing time may be reduced to 8–10 sec where high frequency mixing devices are used, and to 6–8 sec with the Rotomix device.

Filling protection
In the first phase of their 2-phase setting process, glass ionomers are sensitive to moisture. Subsequently, there is the possibility of dehydration which may be prevented by the use of Ketac™ Glaze, manufactured by 3M ESPE, directly after modeling. Apply Ketac Glaze to all exposed surfaces of the glass ionomer cement and cure with light for 10 sec.

Finishing
Beginning no earlier than 7 minutes after the start of mixing, use Arkansas stone, fine-grain diamonds, polishing discs of decreasing grain size or silicone polishers for finishing. If desired, apply Ketac Glaze again for surface sealing, and cure with light for 10 sec.

Incompatibilities
In susceptible individuals, sensitization to the product cannot be excluded. Use of the product should be discontinued, if allergic reactions are observed and the product is to be completely removed. In rare cases, complications of the pulp may occur, especially if the recommendations for processing are disregarded.

Storage and Shelf-Life
Do not store the product above 25°C/77°F. Do not use after the expiry date.

Customer Information
No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty
3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability
Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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