3M ESPE Masking Agent, manufactured by 3M ESPE, has been designed for those restorations where greater masking is required than can be obtained with a 3M ESPE restorative shade alone. It is indicated for use as an undercoating to mask enamel imperfections and discolorations prior to veneering enamel surfaces. It is also used to mask pins and exposed metals as in porcelain repair and cosmetic covering of mesial amalgams.

To enhance marginal sealing and provide retention, Masking Agent should be used with the acid etch technique and application of an Adper™ Scotchbond™ adhesive.

Shades available: Universal: B3; White.

Precautions for Dental Personnel and Patients: Masking Agent Precautions

Masking Agent contains BIS-GMA and TEGDMA Resins. A small percentage of the population is known to have an allergic response to acrylate resins. To reduce the risk of allergic response, minimize exposure to these materials. In particular, exposure to uncured resins should be avoided. If accidental contact with eyes or prolonged contact with oral soft tissue occurs, flush with large amounts of water. If skin contact occurs, wash skin with soap and water.

Step 1 - Tooth Preparation

- Prophy the enamel surface(s) to be etched using pumice and water. Do not use prophy pastes containing oils or fluorides.
- Rinse thoroughly with water and dry.
- Choose the appropriate shade that will blend with the surrounding tooth structure.

Step 2 - Etching

Use a matrix strip to protect adjacent tooth surface(s) during etching. Etch according to the instructions of the Adper Scotchbond adhesive product that is used.

- If metal surfaces or fractured porcelain are to be masked, these surfaces are also treated with the etchant for cleaning. Do not contaminate from this point on. If contamination occurs, re-etch.

Dry with oil-free, moisture-free air.

Step 3 - Adhesive Application

Apply and cure the adhesive according to the instructions of the Adper Scotchbond adhesive used. In the case of porcelain repair, RelyX™ Ceramic Primer, manufactured by 3M ESPE, should be placed on the porcelain surface and exposed metal prior to the placement of the adhesive.

Step 4 - Application

- Dispense the required amount of Masking Agent onto pad and immediately replace the syringe cap.
- Place the selected shade of Masking Agent immediately and brush a thin layer (0.25 - 0.5 mm) over the areas to be masked. Cure 20 seconds. Additional layers of Masking Agent may be added to mask about 75% of the stain (cure each layer 20 seconds).

If tints are to be used, a thin coat of the appropriate shade is placed over the Masking Agent and cured 20 seconds. Refer to the appropriate instructions.

Step 5 - Completion of Restoration

- Complete the restoration with an appropriate shade of restorative. See appropriate instructions.
- Finish the restoration with the Sof-Lex™ Disc System, manufactured by 3M ESPE.

Storage and Use

- Do not expose materials to elevated temperature or intense light.
- Do not store materials in proximity to eugenol-containing products.
- This system is designed to be used at room temperature of approximately 21- 24° C or 70- 75° F.
- Shelf life at room temperature is 36 months. See outer package for expiry date.

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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3M, ESPE, Adper, Scotchbond, RelyX and Sof-Lex are trademarks of 3M or 3M ESPE AG. © 3M 2005. All rights reserved. 44-0023-3950-3-B