Aplicap™/Maxicap™ System

Activator and Applier for Capsules

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Made in Germany by
3M ESPE AG
 Dental Products
 D-82229 Seefeld - Germany

3M ESPE
 Dental Products
 St. Paul, MN 55144-1000

3M, ESPE, Aplicap and Maxicap are trademarks of 3M or 3M ESPE AG.
 © 3M 2003. All rights reserved.


44000129-08/01

ENGLISH


Product Description

The Aplicap and Maxicap System, manufactured for 3M ESPE, each contain an activator for capsule activation, and an applier for application of the ready-mixed capsule contents. The Aplicap activator and applier are used to process Aplicap capsules. The activator is equipped with a red plastic base, and the applier contains a red marking at the top of the handpiece. The Maxicap activator and applier are used to process Maxicap capsules. The activator is equipped with a blue plastic base, and the applier contains a blue marking at the top of the handpiece.


Aplicap Capsule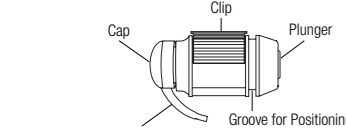


Maxicap Capsule

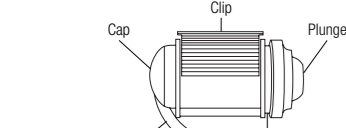Activator

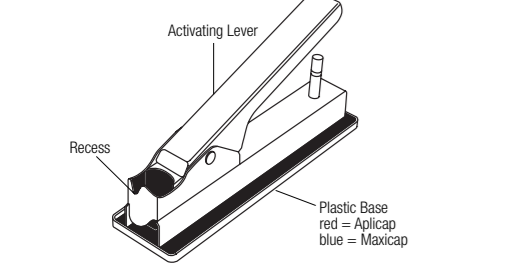Applier

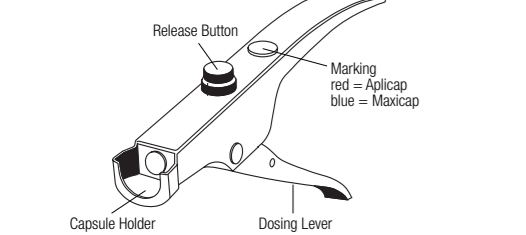
☞ Instructions for Use should not be discarded for the duration of product use.

Areas of Application


- Activation and application of Aplicap capsules
- Activation and application of Maxicap capsules

Application

- Place the activator on a level and sturdy surface.
- Insert the capsule into the activator with the application tip pointing downward, such that the tip rests in the recess.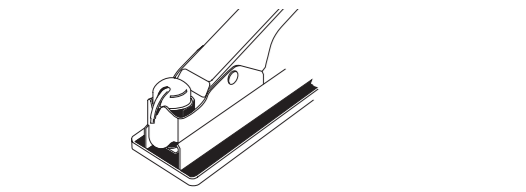


- **Slowly depress the activating lever down to the stop, and hold for 2 to 4 seconds. It is essential to strictly comply with this procedure to prevent premature curing of the material inside the capsule.**
- Slowly depressing the lever ensures that the capsule does not get stuck inside the activator and that the clip above the liquid reservoir engages in the proper position. Keeping the lever depressed for the indicated time ensures that all of the liquid is squeezed from the reservoir into the mixing chamber containing the powder.

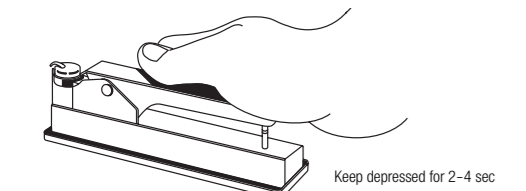
- Subsequently, insert the capsule into a mixing unit and mix the capsule contents as recommended in the respective Instructions for Use.


- Directly after mixing, insert the capsule into the applicator such that the capsule holder engages the groove at the plunger end of the capsule and the capsule is pressed into the capsule holder as far as it will go. The capsule can be rotated inside the holder to achieve an optimal position for application.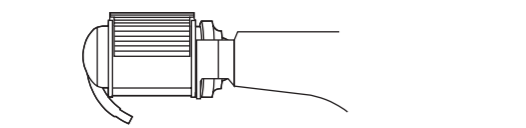

- Activate the dosing lever two or three times. This pushes the plunger into the capsule and moves the ready-mixed paste toward the application tip.
- Push the application tip up as far as it goes to open its orifice. The paste is not transported if the orifice is only half-open.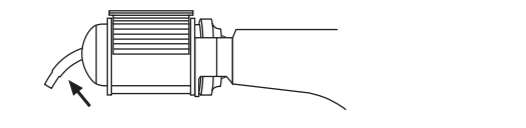

- To apply the paste, slowly activate the dosing lever.
- To remove the empty capsule from the applicator, push the release button, while the dosing lever is not activated. This retracts the plunger and releases the capsule.

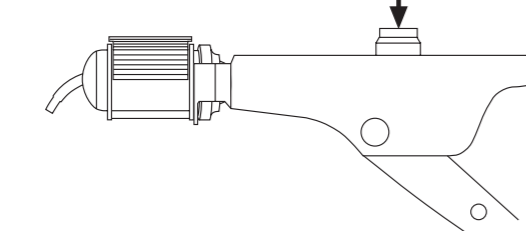Cleaning

- Clean the contaminated activator/applier with standard cleaning agents.
- All parts of the activator/applier can be cleaned with standard alcohol-based spray disinfectants.
- All metal parts can be autoclaved at a maximum temperature of 140°C/285°F.
- Remove the plastic base of the activator prior to autoclaving as this part can not be autoclaved.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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