3M ESPE MiniTip™ Application System

- D** Applikationssystem
- F** Système d'application
- I** Sistema di applicativo
- E** Sistema de aplicación
- P** Sistema de aplicação
- NL** Aanbrengsysteem
- GR** Σύστημα Εφαρμογής
- S** Appliceringssystem
- FIN** Annostelujärjestelmä
- DK** Appliceringssystem
- N** Applikasjonssystem

Instructions for Use Gebrauchsinformation Οδηγίες χρήσης Bruksanvisning Käyttöohje Informazioni per l'uso Informacion de uso Instruções de uso

3M ESPE AG
Dental Products
D-82229 Seefeld - Germany

3M ESPE
Dental Products
St. Paul, MN 55144-1000

3M ESPE Technical Hotline/MSDS Information 1-800-634-2249.

3M, ESPE, Alkaliner, Impresept and MiniTip are trademarks of 3M or 3M ESPE AG.
© 3M 2006. All rights reserved.

ENGLISH

Product Description


The MiniTip application system, manufactured by 3M ESPE, allows precise, accurate application and sparing use. Accidental spattering of the paste is not possible with this dosing system. The use of direct application tips is not necessary for every product.

These instructions for use must be kept for as long as the product is used.

Area of Use

- The dosing and application of products in MiniTip cartridges.

Installation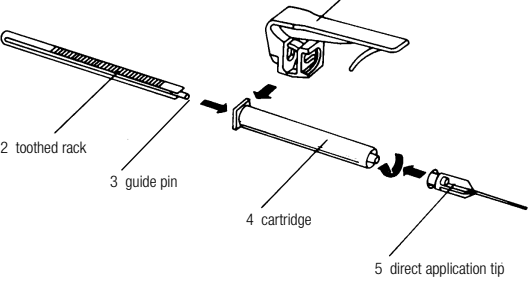


- Slide the dosing lever (1) sideways onto the cartridge (4) until it stops.
- Slide the toothed rack (2) into the dosing lever with the teeth up and the rounded side to the back so that the guide pin (3) slides into the hole of the rubber piston in the cartridge (4).
- Twist the cap off of the cartridge (4). Do not discard the cap.
- To apply Etching Gel MiniTip™, manufactured by 3M ESPE, mount a direct application tip onto the cartridge (5) and lock it into place by turning it one quarter-turn clockwise. No direct application tip is necessary to dose Alkaliner™ MiniTip™, manufactured by 3M ESPE.

Application

- The direct application tip can be bent as necessary for application.
- Apply the paste by pressing the dosing lever lightly.
- Attention: The direct application tip must be discarded after use and a new direct application tip used for each new patient for hygienic reasons. Seal the cartridge with the original cap after every use.**

Disassembly

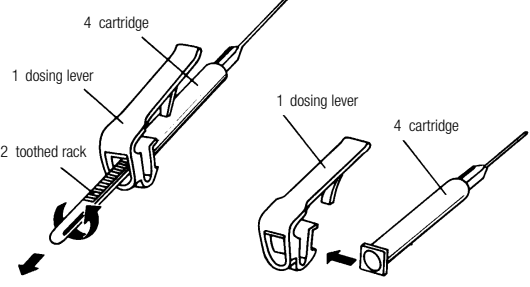- All parts must be disassembled to replace an empty cartridge with a full cartridge or to clean the MiniTip application system:
- Turn the toothed rack (2) by 90° and pull it out of the cartridge (4) and the dosing lever (1).
- Pull the dosing lever (1) sideways from the cartridge (4).

Cleaning

- The MiniTip application system without the cartridge can be immersed in a standard disinfecting solution such as Impresept™, manufactured for 3M ESPE, after it has been disassembled. Follow manufacturer's instructions for the duration of immersion, e.g., 10 min for Impresept. Impresept is not available in all countries.
- Disinfect the cartridge (4) after sealing it with the original cap using a spray or wipe disinfectant.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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