<div>3M ESPE</div> <div>Polyether Adhesive</div>
Adhesive
 D Adhäsiv
 F Adhésif
 I Adesivo
 E Adhesivo
 P Adesivo
 NL Adhesief
 GR Συγκολλητικός παράγοντας
 S Adhesiv
 FIN Sidosaine
 DK Adhæсив
 N Adhesiv

<div> <div>Instructions for Use</div> <div>Gebrauchsinformation</div> <div>Mode d'emploi</div> <div>Informazioni per l'uso</div> <div>Información de uso</div> <div>Instruções de uso</div> </div>	<div> <div>Gebbruksinformatie</div> <div>Οδηγίες χρήσης</div> <div>Bruksanvisning</div> <div>Käyttöinfo</div> <div>Brugsanvisning</div> <div>Bruksinformatasjon</div> </div>
--	--

Made in Germany by **3M ESPE AG**
Dental Products
D-82229 Seefeld – Germany

3M ESPE
Dental Products
St. Paul, MN 55144-1000
3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.

3M and ESPE are trademarks of 3M or 3M ESPE AG.
© 3M 2004. All rights reserved.

--	--

ENGLISH

Product Description
Polyether Adhesive, manufactured by 3M ESPE, affords firm adhesion between impression trays and polyether impression materials (not suitable for use with polyvinyl siloxane impression materials).

☞ Instructions for Use should not be discarded for the duration of product use.

Areas of Application

- All metal trays and custom methacrylate trays.

Application

- With the brush apply a thin layer of adhesive to the tray. Allow to dry completely (min. 30–60 sec).
- Longer drying times (ideally 15 min) provide for stronger adhesion.
- Carefully re-cap the bottle after use. Remove any adhesive clinging to the threads on the bottle and the screw cap before re-capping using an acetone soaked cloth.

Cleaning

- Residual impression or adhesive material is easy to remove from the tray with acetone.

Notes

- Over-filled adhesive-containing bottles (by adding residual quantities from other bottles) may burst even when heated only slightly.
- Polyether Adhesive may not adhere satisfactorily to custom trays made from other plastics or base plates.
- Application of adhesive to textile materials, plastic panels, varnished surfaces, etc., should be avoided as it may not be possible to completely remove the adhesive.
- Use Polyether Adhesive in well-ventilated areas only.
- Rinse exposed skin with water.

Incompatibilities

In susceptible individuals, sensitization to the product cannot be excluded. If allergic reactions occur, use of the product should be discontinued and the product completely removed.

Storage and Stability

Store the product at 15–25°C/59–77°F. Do not use after the expiry date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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